

CASSETTE AND DIGITAL IMAGING HOLDER

INSTRUCTION GUIDE

FOR DIGITAL IMAGING:

The cassette holder holds wet cassettes in the slots on the front side. It is now also compatible with new digital imaging technology. To use the cassette holder for digital imaging, turn it over and attach enclosed Velcro pads to back panel. Then attach Velcro to film enclosure.

FOR MOUNTING HARDWARE:

The cassette holder mounting hardware mounts onto your existing disruper mount. Simply unscrew wing nuts and switch the disruper clamps with the V-blocks. The T-bracket can be screwed onto the top of the cassette holder (with two 6-32 screws). Then clamp the pole and attach T-bracket to the pole with 5/16" wing stud. Cassette can be raised or lowered to desired height.

FOR MULTIPLE CASSETTE HOLDERS:

The cassette holders can be stacked on top of each other for larger x-ray imaging. To stack, simply remove feet and slide screws into slots on the bottom of the cassette holder.



HARTZELL MACHINE WORKS, INC.

Phone: 610-485-3502 Fax: 610-485-7471

E-mail: HMWINC@aol.com